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80% Neoprene, 20% Nylon  
Spacer Fabric: 45% Polyester, 55% Spandex

EN | English

A-107, Lumbo Belt

## INSTRUCTION MANUAL

### INDICATIONS

Lumbar Support for the orthotic fitting of the lower back. Recommended for the treatment of painful conditions of the lumbar spine. Lumbar sciatica, spondylosis, mild spondylosis, spondyloarthritis, lumbar facet syndrome, lower back pain.

### FIT

Wrap the belt around your abdomen, make sure it is positioned on the centre of your back and secure the Velcro fasteners. Use the additional tension bands to adjust to the tightness that suits your own needs.

In addition to measuring the waist, the belt should be tried on to ensure a good fit and to understand how to put it on properly. Tighten the belt based on the intensity of the pain and type of activity.

### CAUTIONS

Do not use the product if you are allergic to one of its components. In case of any irritation occurrence, discontinue the use of the product and seek medical advice.

### CARE AND MAINTENANCE

Hand wash in warm soapy water (maximum 30°C). Dry flat, away from sources of excessive heat (radiator, tumble dryer).

#### Material Composition

80% Neoprene, 20% Nylon  
Spacer Fabric: 45% Polyester, 55% Spandex