



ΘΕΟΔΩΡΟΠΟΥΛΟΣ Α. ΙΩΑΝΝΗΣ  
Sanaflex Anatomic  
Κυπρου 93, Athens, 164 51



A-127L / A-127R  
Νάρθηκας Καρπού / Wrist Support

## ΜΕΓΕΘΟΛΟΓΙΟ

One Size  
(S - XL)

GR | Ελληνικά

A-127L / A-127R, Νάρθηκας Καρπού

## ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΗΣ

### ΕΝΔΕΙΞΕΙΣ

Τενοντίτιδες, αρθροπάθεια καρπού, πόνος, αστάθεια, επακόλουθα τραυματισμών, προληπτικά.

### ΤΟΠΟΘΕΤΗΣΗ

Περάστε τον αντίχειρα μέσα από την οπή. Τυλίξτε τη δέστρα γύρω από τον καρπό και ασφαλίστε με το velcro, προσαρμόζοντας την ασκούμενη πίεση στις ανάγκες σας. Η μεταλλική μπανέλα παρέχει στήριξη στον καρπό διατηρώντας τον σε λειτουργική θέση σε γωνία 35 μοιρών και μπορεί να αφαιρεθεί.

### ΑΝΤΕΝΔΕΙΞΕΙΣ

Μη χρησιμοποιείτε το προϊόν αν είστε αλλεργικός/η σε κάποιο από τα συστατικά του. Σε περίπτωση εμφάνισης ερεθισμού, διακόψτε τη χρήση του προϊόντος και συμβουλευτείτε το γιατρό σας.

### ΣΥΝΤΗΡΗΣΗ

Πλύσιμο στο χέρι σε νερό στους 30°C με ήπιο σαπούνι ή απορρυπαντικό. Ξεπλύνετε καλά, μην σίβετε, αφήστε να στεγνώσει, μη σιδερώνετε.

#### Σύνθεση

80% Neoprene, 20% Nylon  
Spacer Fabric: 45% Polyester, 55% Spandex



EN | English

A-127L / A-127R, Wrist Support

## INSTRUCTION MANUAL

### INDICATIONS

Wrist splint, with an adjustable and removable aluminum stay, pain relief through immobilization of the wrist and hand in a functional position. For the treatment of Tendinitis, osteoarthritis of the wrist, side effects of injuries, neurological, pain, instability, etc.

### FIT

Insert your thumb through the hole. Wrap the bandage around the wrist and secure it, adapting the pressure to your needs. The removable metal band provides support to the wrist by keeping it in a functional position at an angle of 35 degrees.

### CAUTIONS

Do not use the product if you are allergic to one of its components. In case of any irritation occurrence, discontinue the use of the product and seek medical advice.

### CARE AND MAINTENANCE

Hand wash in warm soapy water (maximum 30°C). Dry flat, away from sources of excessive heat (radiator, tumble dryer).

#### Material Composition

80% Neoprene, 20% Nylon  
Spacer Fabric: 45% Polyester, 55% Spandex

