



ΘΕΟΔΩΡΟΠΟΥΛΟΣ Α. ΙΩΑΝΝΗΣ
Sanaflex Anatomic
Κυρου 93, Athens, 164 51



A-128
Καρποδαχτυλικός Νάρθηκας / Thumb Support

ΜΕΓΕΘΟΛΟΓΙΟ

One Size
(S - XL)

GR | Ελληνικά

A-128, Καρποδαχτυλικός Νάρθηκας

ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΗΣ

ΕΝΔΕΙΞΕΙΣ

Σταθεροποίηση της βάσης του καρπού και του αντίχειρα με τη βοήθεια της ελαστικής αλουμινένιας ράγας. Διευκολύνει τον πρηνισμό-υππιασμό. Ριζορθοπάθεια, χρόνιος πόνος, αποφυγή καταπόνησης αρθρώσεων και βοηθητική υποστήριξη στις καθημερινές δραστηριότητες.

ΤΟΠΟΘΕΤΗΣΗ

Τοποθετήστε τον αντίχειρα σας σε αναπαυτική θέση στην κορυφή της προέκτασης με την μεταλλική λάμα και ασφαλίστε με το velcro. Στη συνέχεια τυλίξτε τη δέστρα γύρω από τον καρπό και ασφαλίστε με το velcro. Κάντε διορθώσεις μετά την τοποθέτηση προσαρμόζοντας την ασκούμενη πίεση στις ανάγκες σας. Η μεταλλική μπανέλα είναι εύκαμπτη και μπορείτε να ρυθμίσετε την κλίση της.

ΑΝΤΕΝΔΕΙΞΕΙΣ

Μη χρησιμοποιείτε το προϊόν αν είστε αλλεργικός/η σε κάποιο από τα συστατικά του. Σε περίπτωση εμφάνισης ερεθισμού, διακόψτε τη χρήση του προϊόντος και συμβουλευτείτε το γιατρό σας.

ΣΥΝΤΗΡΗΣΗ

Πλύσιμο στο χέρι σε νερό στους 30°C με ήπιο σαπούνι ή απορρυπαντικό. Ξεπλύνετε καλά, μην στίβετε, αφήστε να στεγνώσει, μη σιδερώνετε.

Σύνθεση

80% Neoprene, 20% Nylon
Spacer Fabric: 45% Polyester, 55% Spandex



EN | English

A-128, Thumb Support

INSTRUCTION MANUAL

INDICATIONS

Stabilization of the base of the thumb and wrist with the help of a flexible aluminium splint. Encourages pronation and supination. Basal thumb arthritis, joint load reduction, and support in everyday and work tasks.

FIT

Place your thumb in a comfortable position on top of the extension with the metal blade and secure with the velcro strap. Following the previous step wrap the binding around the wrist and secure with velcro. Make adjustments after installation by adapting the pressure to your needs. The metal bar is flexible and you can adjust its inclination.

CAUTIONS

Do not use the product if you are allergic to one of its components. In case of any irritation occurrence, discontinue the use of the product and seek medical advice.

CARE AND MAINTENANCE

Hand wash in warm soapy water (maximum 30°C). Dry flat, away from sources of excessive heat (radiator, tumble dryer).

Material Composition

80% Neoprene, 20% Nylon
Spacer Fabric: 45% Polyester, 55% Spandex

