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## INSTRUCTION MANUAL

### INDICATIONS

- Self adherent tape for sports injuries.
  - Sticks to itself – not to skin or hair.
  - Excellent conformability
- Cohesive flexible bandage.
- Sticks to itself – not to skin or hair.
- Excellent conformability.
- Firmly secures dressing on wounds.
- Controlled compression – will not constrict.
- Leaves no sticky residue on skin.

### APPLICATION

Clean and dry skin, then apply Self-Adherent Wrap. Hand-tear or cut bandage to desired length. Press down and seal the end of the bandage with your fingers, meshing the layers together to provide long staying power. Can also be used for compression and support for ligaments, tendons and muscles.

### CAUTIONS

- Use the product for the mentioned indication only.
- For single person use only
- Do not use the product if you are allergic to one of its components.
- If you experience pain, swelling, or sensation changes as a result of wearing this item, stop use immediately and consult your healthcare provider.
- Avoid tight application as it could restrict circulation.
- If the wrap feels too tight, remove it and wrap again.

### Material Composition

95% PP, 5% Spandex