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**Material Composition**  
95% Cotton, 5% Elastane

EN | English

K100, Kinesiology Tape

## INSTRUCTION MANUAL

### INDICATIONS

Ideal for sport.

For pain relief and support of Shoulder, neck, tennis elbow, back, hamstring, knee, calf, foot. High quality cotton fabric that maintains strength and stability.

Extra strong adhesive to insure performance in almost any condition.

Breathable fabric for sweat management.

### APPLICATION

Cut corners using sharp scissors in a round shape to minimize peeling.

This product is not intended to replace professional medical treatment. It is recommended to consult with a professional before use

Use the product for the mentioned indication only.

Keep out of reach of children.

Do not use the product if you are allergic to one of its ingredients.

People with sensitive skin are advised to try a small piece of the tape on their skin for 24 hours before use.

### CAUTIONS

Use the product for the mentioned indication only. Keep out of reach of children. Do not use the product if you are allergic to one of its components. Individuals with sensitive skin are advised to try a small piece of the tape on their skin for 24 hours before use.

Avoid tight application as it could restrict circulation. If the wrap feels too tight, remove it and wrap again. Stop using the product in case of soreness, redness or irritated skin. Not suitable for pregnant women or people diagnosed with cancer, as it may interfere with treatment. Do not use when suffering from Deep Vein Thrombosis (DVT), kidney failure or congestive heart failure (CHF).

**Material Composition**  
95% Cotton, 5% Elastane